


How to Connect


Device Activation

- Scan QR code for Device Activation or you can manually input the Serial Number inside the app.
- This feature provides additional guarantee that the device and program are synchronized and would provide accurate analysis results.

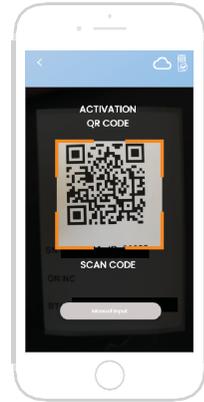

1 Attaching Magnetic Strip

- Put the paper film guide properly on the back of the phone and direct the arrow part to fit the camera lens.
- Firmly attach the magnetic strip on the provided space in the middle and press it for 10 seconds to secure its attachment.

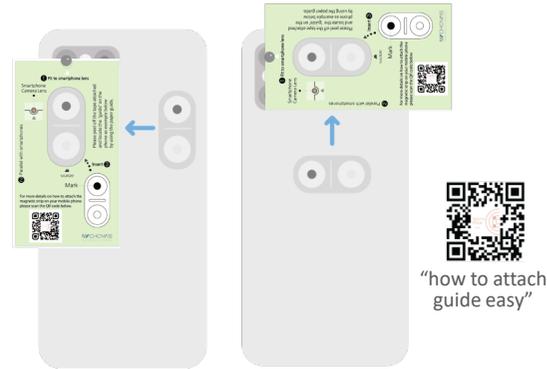

2 Attaching Device to Phone

- Attach the dpi Harris device on the magnetic strip.
- You can now start using the DermoPico Hair Application.



3 Turn On Device

- Press power switch button of the dpi Harris device for 3 seconds or until blue LED light blinks.



This device will automatically go to sleep mode after 10 minutes if not used.


4 Turn On Bluetooth

- Turn on the phone's Bluetooth to connect the app to the device.



5 Connecting Device to Phone

- Open DermoBella App.
- Login or Sign up if no account is registered.
- After login, it will automatically scan devices that are available in range.
- Choose BLE ID of device to connect.

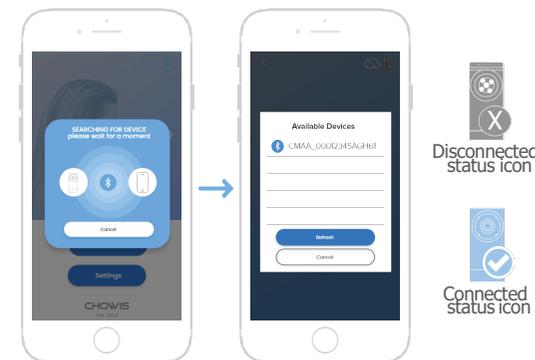

6 Calibrating Device

- After connecting to device, notice will pop up and tap "Proceed" to calibrate device.
- Make sure to align the mobile camera with the calibration paper lines and dot to keep mobile camera in focus.

